

Living Donor

SPECIMEN GUIDE



| Test Number | Test Name | Specimen Volume | Preferred Specimen Type(s) | Preferred Collection Tube(s) | Specimen Viability |
|-------------|--|---|---|--|---|
| 138662 | Cytomegalovirus Total Antibody | 0.2 mL | Plasma | EDTA plasma (lavender-top) | 7 days at refrigerated temperature or freeze |
| 138697 | Cytomegalovirus Total With Reflex to IgM | 0.2 mL | Plasma and serum | EDTA plasma (lavender-top) and serum separator (speckled-top) or plain (red-top) | 7 days at refrigerated temperature or freeze |
| 138930 | Cytomegalovirus Total With Reflex to IgM and IgG | 0.2 mL | Plasma and serum | EDTA plasma (lavender-top) and serum separator (speckled-top) or plain (red-top) | 7 days at refrigerated temperature or freeze |
| 138828 | <i>C trachomatis</i> / <i>N gonorrhoeae</i> NAA | 2 mL urine in APTIMA® urine specimen transport tube 1 APTIMA® swab | APTIMA® urine specimen transport tube APTIMA® swab | APTIMA® urine specimen transport tube (Raw urine must be transferred to collection transport tube within 24 hours of collection.) 1 urethral swab (male) in APTIMA® unisex swab collection kit 1 endocervical swab (female) in APTIMA® unisex swab collection kit. | APTIMA® urine: Within 30 days at room temperature or refrigerated temperature (2°C - 30°C). Additionally, urine can be stored up to 12 months at -20°C to -70°C. APTIMA® swab: Within 60 days at room temperature or refrigerated temperature (2°C - 30°C). Additionally, swab can be stored up to 12 months (6 months for urethral swab) at -20°C to -70°C. |
| 138699 | Hepatitis B Core Antibody, Total | 2 mL | Serum Plasma | Plain (red-top) Serum separator (speckled-top) EDTA plasma (lavender-top) | 30°C or colder for up to 7 days, 14 days at 2°C - 8°C or freeze |
| 138979 | Hepatitis B Core Total With Reflex to IgM | 2 mL | Serum Plasma | Plain (red-top) Serum separator (speckled-top) EDTA plasma (lavender-top) | 30°C or colder for up to 7 days, 14 days at 2°C - 8°C or freeze |
| 138874 | Hepatitis B Surface Antigen | 2 mL | Serum Plasma | Plain (red-top) Serum separator (speckled-top) EDTA plasma (lavender-top) | 30°C or colder for up to 7 days, 14 days at 2°C - 8°C or freeze |
| 138749 | Hepatitis B Surface Antigen With Reflex to Confirmatory Neutralization | 2 mL | Serum Plasma | Plain (red-top) Serum separator (speckled-top) EDTA plasma (lavender-top) | 30°C or colder for up to 7 days, 14 days at 2°C - 8°C or freeze |
| 138907 | Hepatitis C Antibody | 2 mL | Serum Plasma | Plain (red-top) Serum separator (speckled-top) EDTA plasma (lavender-top) | 30°C or colder for up to 7 days, 14 days at 2°C - 8°C or freeze |
| 138948 | HIV-1/HIV-2 PLUS O Antibody Screen | 2 mL | Serum Plasma | Plain (red-top) Serum separator (speckled-top) EDTA plasma (lavender-top) | Living donor specimens at 2°C - 8°C or frozen for up to 14 days or 30°C or colder for up to 7 days (inclusive of shipping time). Storage at a combination of these temperatures may not exceed 14 days. |
| 138931 | HIV-1/HIV-2 PLUS O Antibody Screen With Reflex to HIV-1 Western Blot | 2 mL | Serum Plasma | Plain (red-top) Serum separator (speckled-top) EDTA plasma (lavender-top) | Living donor specimens at 2°C - 8°C or frozen for up to 14 days or 30°C or colder for up to 7 days (inclusive of shipping time). Storage at a combination of these temperatures may not exceed 14 days. |
| 138981 | HIV-1 Antibody Western Blot | 0.2 mL | Serum Plasma | Plain (red-top) Serum separator (speckled-top) EDTA plasma (lavender-top) | Within 7 days at refrigerated temperature or freeze |
| 139240 | HIV-1/HCV/HBV (Ultrio NAT) | 5 mL whole blood 3 mL plasma | Whole blood Plasma | EDTA whole blood (lavender-top) EDTA plasma (lavender-top) | Whole blood or plasma may be stored up to 72 hours from time of draw at 2°C - 25°C, temperatures not to exceed 30°C are acceptable for no more than 24 hours. Specimens may be stored for an additional 5 days at 2°C - 8°C after centrifugation/separation. Plasma separated from cells may be stored for up to 6 month at ≤-20°C. Do not freeze whole blood. |
| 138922 | HTLV-I/II Antibody | 2 mL | Serum Plasma | Plain (red-top) Serum separator (speckled-top) EDTA plasma (lavender-top) | 30°C or colder for up to 7 days, 14 days at 2°C - 8°C or freeze |
| 138780 | HTLV-I/II Antibody With Reflex to Immunoblot | 2 mL | Serum Plasma | Plain (red-top) Serum separator (speckled-top) EDTA plasma (lavender-top) | Within 7 days at refrigerated temperature or freeze |
| 138984 | HTLV-I/II Immunoblot | 0.5 mL | Serum Plasma | Plain (red-top) Serum separator (speckled-top) EDTA plasma (lavender-top) | Within 7 days at refrigerated temperature or freeze |
| 139280 | Syphilis (<i>T pallidum</i>) IgG | 0.5 mL | Serum | Plain (red-top) Serum separator (speckled-top) | Within 5 days at refrigerated temperature or freeze |
| 139290 | <i>T cruzi</i> (Chagas) Antibody | 2 mL | Serum Plasma | Plain (red-top) Serum separator (speckled-top) EDTA plasma (lavender-top) | Living donor specimens at 2°C - 8°C or frozen for up to 14 days or 30°C or colder for up to 7 days (inclusive of shipping time). Storage at a combination of these temperatures may not exceed 14 days. |

| Test Number | Test Name | Specimen Volume | Preferred Specimen Type(s) | Preferred Collection Tube(s) | Specimen Viability |
|-------------|--------------------------------|---------------------------------|----------------------------|---|---|
| 138896 | West Nile Virus (Procleix NAT) | 4 mL whole blood 2 mL plasma | Whole blood Plasma | EDTA whole blood (lavender-top) EDTA plasma (lavender-top) | Whole blood or plasma may be stored for up to 72 hours from time of draw at 2°C-25°C; temperatures not to exceed 30°C are acceptable for no more than 24 hours. Specimens may be stored for an additional 5 days at 2°C-8°C after centrifugation. Plasma separated from the cells may be stored for up to 9 months at ≤ -20°C or up to 15 months at ≤ -70°C before testing. Do not freeze whole blood. |

Donor Screening Panels

| Test Number | Test Name | Specimen Volume | Preferred Specimen Type(s) | Preferred Collection Tube(s) | Specimen Viability |
|-------------|--|--|---|---|----------------------|
| 139060 | FDA Female, No Reflex HBsAg, HBcore Total, HCV, HIV-1/HIV-2 PLUS O, Syphilis (<i>T pallidum</i>), CT/NG NAA, HIV-1/HCV/HBV NAT | 2.5 mL serum 3 mL plasma 2 mL urine in APTIMA® urine specimen transport tube (fill between the black lines) or 1 APTIMA® swab | Serum Whole blood or plasma Urine or swab | Plain (red-top) Serum separator (speckled-top) EDTA whole blood or EDTA plasma (lavender-top) APTIMA® urine specimen transport tube (Raw urine must be transferred to collection transport tube within 24 hours of collection) or 1 endocervical swab in APTIMA® unisex swab collection kit | See individual tests |
| 139061 | FDA Female, With Reflex HBsAg With Reflex to Neutralization, HBcore Total With Reflex to IgM, HCV, HIV-1/HIV-2 PLUS O With Reflex to HIV-1 WB, Syphilis (<i>T pallidum</i>), CT/NG NAA, HIV-1/HCV/HBV NAT | 3.8 mL serum 3 mL plasma 2 mL urine in APTIMA® urine specimen transport tube (fill between the black lines) or 1 APTIMA® swab | Serum Whole blood or plasma Urine or swab | Plain (red-top) Serum separator (speckled-top) EDTA whole blood or EDTA plasma (lavender-top) APTIMA® urine specimen transport tube (Raw urine must be transferred to collection transport tube within 24 hours of collection) or 1 endocervical swab in APTIMA® unisex swab collection kit | See individual tests |
| 139062 | FDA Male, No Reflex HBsAg, HBcore Total, HCV, HIV-1/HIV-2 PLUS O Syphilis (<i>T pallidum</i>), HTLV-I/II, CMV Total, CT/NG NAA, HIV-1/HCV/HBV NAT | 4.7 mL serum 3.2 mL plasma 2 mL urine in APTIMA® urine specimen transport tube (fill between the black lines) or 1 APTIMA® swab | Serum Whole blood or plasma Urine or swab | Plain (red-top) Serum separator (speckled-top) EDTA whole blood or EDTA plasma (lavender-top) APTIMA® urine specimen transport tube (Raw urine must be transferred to collection transport tube within 24 hours of collection) or 1 urethral swab in APTIMA® unisex swab collection kit | See individual tests |
| 139063 | FDA Male, With Reflex HBsAg With Reflex to Neutralization, HBcore Total With Reflex to IgM, HCV, HIV-1/HIV-2 PLUS O With Reflex to HIV-1 WB, Syphilis (<i>T pallidum</i>), HTLV-I/II With Reflex to Immunoblot, CMV Total With Reflex to IgM, CT/NG NAA, HIV-1/HCV/HBV NAT | 6.0 mL serum 3.2 mL plasma 2 mL urine in APTIMA® urine specimen transport tube (fill between the black lines) or 1 APTIMA® swab | Serum Whole blood or plasma Urine or swab | Plain (red-top) Serum separator (speckled-top) EDTA whole blood or EDTA plasma (lavender-top) APTIMA® urine specimen transport tube (Raw urine must be transferred to collection transport tube within 24 hours of collection) or 1 urethral swab in APTIMA® unisex swab collection kit | See individual tests |
| 139323 | Donor Screening Panel 134 FDA Female With WNV, No Reflex HBsAg, HBcore Total, HCV, HIV-1/HIV-2 PLUS O, Syphilis (<i>T pallidum</i>), CT/NG NAA, HIV-1/HCV/HBV NAT, WNV NAT | 2.5 mL serum 4 mL plasma 2 mL urine in APTIMA® urine specimen transport tube (fill between the black lines) or 1 APTIMA® swab | Serum Whole blood or plasma Urine or swab | Plain (red-top) Serum separator (speckled-top) EDTA whole blood or EDTA plasma (lavender-top) APTIMA® urine specimen transport tube (Raw urine must be transferred to collection transport tube within 24 hours of collection) or 1 endocervical swab in APTIMA® unisex swab collection kit | See individual tests |
| 139472 | FDA Female With WNV, With Reflex HBsAg With Reflex to Neutralization, HBcore Total With Reflex to IgM, HCV, HIV-1/HIV-2 PLUS O With Reflex to HIV-1 WB, Syphilis (<i>T pallidum</i>), CT/NG NAA, HIV-1/HCV/HBV NAT, WNV NAT | 3.8 mL serum 4 mL plasma 2 mL urine in APTIMA® urine specimen transport tube (fill between the black lines) or 1 APTIMA® swab | Serum Whole blood or plasma Urine or swab | Plain (red-top) Serum separator (speckled-top) EDTA whole blood or EDTA plasma (lavender-top) APTIMA® urine specimen transport tube (Raw urine must be transferred to collection transport tube within 24 hours of collection) or 1 endocervical swab in APTIMA® unisex swab collection kit | See individual tests |

Refrigerated specimens are to be maintained at 2°C – 8°C and frozen specimens at minus 20°C or colder unless otherwise specified. Unless explicitly stated otherwise in the package insert, frozen samples that have been stored longer than 6 months after their collection date will not be accepted.

Effective January 2017

For current specimen requirements, please call ViroMed Account Management at 800-582-0077 or e-mail ViroMed_AcctMgmt@labcorp.com.

| Test Number | Test Name | Specimen Volume | Preferred Specimen Type(s) | Preferred Collection Tube(s) | Specimen Viability |
|-------------|--|--|---|---|----------------------|
| 139496 | FDA Male With WNV, With Reflex HBsAg With Reflex to Neutralization, HBcore Total With Reflex to IgM, HCV, HIV-1/HIV-2 PLUS O With Reflex to HIV-1 WB, Syphilis (<i>T pallidum</i>), HTLV-I/II With Reflex to Immunoblot, CMV Total With Reflex to IgM, CT/NG NAA, HIV-1/HCV/HBV NAT, WNV NAT | 6.0 mL serum 4.2 mL plasma 2 mL urine in APTIMA® urine specimen transport tube (fill between the black lines) or 1 APTIMA® swab | Serum Whole blood or plasma Urine or swab | Plain (red-top) Serum separator (speckled-top) EDTA whole blood or EDTA plasma (lavender-top) APTIMA® urine specimen transport tube (Raw urine must be transferred to collection transport tube within 24 hours of collection) or 1 endocervical swab in APTIMA® unisex swab collection kit | See individual tests |
| 139506 | FDA Male With WNV, No Reflex HBsAg, HBcore Total, HCV, HIV-1/HIV-2 PLUS O Syphilis (<i>T pallidum</i>), HTLV-I/II, CMV Total, CT/NG NAA, HIV-1/HCV/HBV NAT, WNV NAT | 4.7 mL serum 4.2 mL plasma 2 mL urine in APTIMA® urine specimen transport tube (fill between the black lines) or 1 APTIMA® swab | Serum Whole blood or plasma Urine or swab | Plain (red-top) Serum separator (speckled-top) EDTA whole blood or EDTA plasma (lavender-top) APTIMA® urine specimen transport tube (Raw urine must be transferred to collection transport tube within 24 hours of collection) or 1 endocervical swab in APTIMA® unisex swab collection kit | See individual tests |

ViroMed provides a validated shipping kit for specimens being transported to ViroMed for testing.



ViroMed Laboratories
1447 York Court
Burlington, NC 27215

800-582-0077
ViroMed_AcctMgmt@LabCorp.com
www.ViroMed.com